

Core Post

Glass/Carbon Fiber Post



Den-Mat's **Core-Posts** are available in two forms, one made from carbon fiber and one from glass fiber. **Core-Posts** exhibit high fatigue and tensile strengths and have a modulus of elasticity comparable to dentin. A post and core build-up restoration is created in one easy application using **Core Paste**, **Tenure®**, and the **Core-Post**.

DIRECTIONS

1. Perform a routine preparation of the root canal. Gates Glidden drills are recommended to remove gutta percha and enlarge the upper portion of the canals (use Gates Glidden drills # 4 and # 5 for **Core-Post** 1.0 mm and 1.2 mm; Gates Glidden drills # 5 and # 6 for **Core-Post** 1.4 mm, 1.7 mm and 2.0 mm).
2. Measure the canal length and then select the post that fits best and still allows room for **Core Paste**.
3. Etch the root canal and the preparation.
4. Rinse thoroughly and gently air dry.
5. Apply **Tenure Bonding System** to the prep. It is not necessary to apply a bonding agent to the post.
6. Fill the prep with **Core Paste** or **Core Paste Syringeable**.
7. Seat the **Core-Post** immediately and finish the core build-up.
 - If dual-cure **Core Paste** is used – light cure the occlusal, facial and the lingual surface, starting with:
 - 5 seconds—**Sapphire™ Plasma Arc Curing Light**
 - 10 seconds—**Allegro High-Intensity LED** or the **Phase II High-Speed Halogen Light**
 - 30 seconds—Conventional curing light.
 - If self-cure **Core Paste** is used – allow the material to set according to directions given.
8. Perform a routine prep for the final restoration.

LIMITED WARRANTY

Den-Mat will replace, or refund the purchase price, of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price, shall be the exclusive remedy for the user. Den-Mat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use, its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

TELEPHONE INQUIRY SERVICE

Technical consultation is available by calling toll-free in the U.S.A. and Canada.

Ordering, inquiry, customer service, technical(800) 445-0345

Domestic fax ordering and inquiry(805) 349-2600

International inquiry(805) 347-7990

International fax ordering and inquiry(805) 349-2607

Websitewww.denmat.com



753362541W

DenMat
ISO 9001 CERTIFIED

1-800-445-0345