



# Desensitizing Enhancer for Chairside Whitening

Store at Room Temperature

LUMIBRITE™ Professional Chairside Whitening is designed to comfortably and quickly whiten patients' teeth. The combination of LUMIBRITE Desensitizing Enhancer and LUMIBRITE Whitening Gel reduces the chairtime required for effective whitening compared to traditional chairside bleaching treatments, and contains fluoride as an added benefit.\*

The LUMIBRITE Professional Whitening Kit is for in-office use only.

- The **Desensitizing Enhancer** contains strong alkaline agents and can cause irritation to unprotected skin or gingiva. Wear protective gloves and eyewear during preparation and application.
- Avoid contacting soft gingival tissue with the **Desensitizing Enhancer**. Using the **Paint-On Dental Dam** included in the **LUMIBRITE Professional Whitening Kit** will shield gingival tissue from this material.
- Material is not for injection.
- Keep out of the reach of children.
- Follow all instructions and cautions for the **LUMIBRITE Whitening System** found in the **LUMIBRITE Professional Chairside Whitening Kit**.
- **LUMIBRITE Desensitizing Enhancer** is intended for use **ONLY** with the **LUMIBRITE Whitening Gel**.

### INSTRUCTIONS FOR USE:

Remove the **LUMIBRITE Whitening Gel** syringe from the refrigerator and allow it to come to room temperature for about 30 minutes before it is used.

1. Prophyl teeth to remove calculus and extrinsic stains before beginning the whitening treatment.
2. Take "before" photographs and record tooth color using a shade guide arranged in bleaching order.
3. Insert cheek retractors. Have the patient bite down and rest tongue on a tongue block.
4. Air-dry the gingival tissue and teeth.
5. Using the dispensing tip, apply **Paint-On Dental Dam** to the gingiva to isolate gums around the selected teeth. Overlap the dental dam onto the gingival tissue of adjacent teeth about 0.5 mm and light-cure for 1–2 seconds per tip width with a **Sapphire® Supreme Plasma Arc Curing (PAC) Light**.
6. Once the **Paint-On Dental Dam** is cured, apply **Lip Moisturizer** provided in the kit to the lips and mucosa, applying well beyond the vermilion border.
7. Remove the tip from the end of the **LUMIBRITE Desensitizing Enhancer** recappable syringe and dispense the **LUMIBRITE Desensitizing Enhancer** in a very thin (about 0.2 mm) layer on the labial surface of the teeth. Distribute the gel evenly using the brush applicator provided in the kit and allow to sit for one minute. Wipe off any excess. Firmly recap the **LUMIBRITE Desensitizing Enhancer syringe**.
8. Remove the tip from the **LUMIBRITE Whitening Gel** syringe and apply a 1–1.5 mm layer directly onto the labial surface of the teeth. Use the brush applicator to distribute the gel evenly over the teeth treated in Step 7 with the **LUMIBRITE Desensitizing Enhancer**. Agitate to mix **LUMIBRITE Desensitizing Enhancer** and **LUMIBRITE Whitening Gel**.
9. Install the **Sapphire Whitening Crystal** in place of the curing tip on the end of the **Sapphire PAC Light** pistol.
10. Position the **Sapphire Whitening Crystal** close to and directly in front of the teeth, using a combination of the articulating arm and chair adjustments. Center the **Sapphire Whitening Crystal** 1/4 to 1/2 inch from the teeth within the cheek retractor with the outer edges touching (if possible).
11. Press the Optional button on the **Sapphire Supreme PAC Light** keypad, then press 2 to select bleaching mode. The letters bL (bleach) will appear in the Program window, the number 60 will appear in the seconds window and 60 minutes will appear on the pistol handle.

12. Press the Start or pistol activator button to start the bleaching process. The number 60 (minutes) appears in the Program window and seconds begin to count down in the Seconds window. When the **Sapphire Supreme PAC Light** beeps, fifteen (15) minutes have elapsed. Check on the comfort of the patient. After the second beep, 30 minutes have elapsed.
13. Move the **Sapphire Whitening Crystal** away from the teeth. Leave the **Paint-On Dental Dam** in place and vacuum gel from the teeth. Wipe with gauze and rinse the teeth while suctioning. Avoid splatter. Some patients with heavy discoloration or areas of unseen hypocalcification may need another 30-minute session to achieve desired whitening results. Evaluate the results with the patient, if possible.

**Note:** In some cases you may have attained acceptable whitening in the first 30-minute session. If so, a second session is not indicated.

**Note:** In some cases, it may be necessary to peel away the Paint-On Dental Dam to make a shade determination. If the **Paint-On Dental Dam** is removed and another 30-minute application is indicated, remember to reapply the **Paint-On Dental Dam** to protect the gingival tissue.

14. If another 30-minute session is desired, reapply both **Desensitizing Enhancer** and **Whitening Gel** following instructions beginning with Step 7.

**Note:** Be sure to inspect the **Paint-On Dental Dam** for any cracking or lifting before re-applying the **Desensitizing Enhancer** and **Whitening Gel**. If the seal has broken from the tissue, it is recommended to remove and replace the **Paint-On Dental Dam** and begin with Step 4.

15. At the end of the second 30-minute session, vacuum gel from the teeth, wipe away with gauze and rinse the teeth while suctioning. Avoid splatter.

16. Use a shade guide arranged in bleaching order to compare the color of the whitened teeth to the original shade recorded in Step 2 and take "after" photographs.

### STORAGE

Store the **LUMIBRITE Desensitizing Enhancer** at room temperature.

### LIMITED WARRANTY

Den-Mat will replace, or refund the purchase price of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price shall be the exclusive remedy for the user. Den-Mat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

### TELEPHONE INQUIRY SERVICE

Technical consultation is available by calling toll-free in the U.S.A. and Canada.

Ordering, inquiry, customer service, technical.....	(800) 445-0345
LUMINEERS® Smile Design Studio customer service .....	(800) 872-8384
Domestic fax ordering and inquiry .....	(800) 233-6628
International inquiry.....	(805) 347-7990
International fax ordering and inquiry .....	(805) 349-2607
Websites: www.lumineersdds.com	
www.denmat.com	

**MSDS SHEETS AVAILABLE AT [www.denmat.com](http://www.denmat.com)**

\*Canadian version does not contain fluoride.