




## with Fluoride, Refill

U.S. Patent 5,767,170



### Do Not Refrigerate

#### INSTRUCTIONS FOR BONDING TO DENTIN

-  Etch the tooth surface with 37% phosphoric acid etchant for 15 seconds. Thoroughly rinse and gently blot or air-dry the tooth surface.
-  Completely immerse the brush tip in **Tenure Quik with Fluoride**. Using a fully saturated brush tip, apply three consecutive coats to the prepared tooth. Allow **Tenure Quik with Fluoride** to sit on the tooth for 15 seconds.
- Remove the solvent from the **Tenure Quik with Fluoride** by directing a gentle stream of dry, oil-free air from the air syringe. The force of the air should be strong enough to extract the adhesive's solvent, but not so forceful as to push the adhesive from the preparation. If the surface is not glossy after drying, reapply **Tenure Quik with Fluoride** and gently re-dry.
-  Light cure **Tenure Quik with Fluoride**, starting with:
  - 5 seconds—**Sapphire™ Plasma Arc Curing Light**
  - 10 seconds—**Allegro High-Intensity LED** or the **Phase II High-Speed Halogen Lights**
  - 30 seconds—Conventional curing light

Apply the restorative material as directed by the manufacturer.

#### INSTRUCTIONS FOR ROOT/DENTIN DESENSITIZATION

- Clean the sensitive area with **Porcelain Polishing Paste**.
- Rinse with warm water and blot dry with a cotton pellet.
- Apply multiple coats (4-6) of **Tenure Quik with Fluoride** to the surface. Air dry with a syringe for several seconds to remove the solvent. Light cure **Tenure Quik with Fluoride** as directed in Step 4 of "Instructions for Bonding to Dentin".
- Repeat Step 3.
- For longer lasting desensitization, apply a thin layer of restorative material (for example, **Virtuoso® Flowable**) on top of the **Tenure Quik With Fluoride**.

#### STORAGE

**Do not** freeze.

**Do not** refrigerate.

**Do not** expose to temperatures exceeding 80° F (27° C).

**Do not** expose to direct sunlight.

Recommended temperature when used is 70° – 75° F (21° – 24° C).

Cool storage is recommended in tropical climates.

Bring to room temperature prior to use.

#### LIMITED WARRANTY

Den-Mat will replace, or refund the purchase price, of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price, shall be the exclusive remedy for the user. Den-Mat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use, its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

#### TELEPHONE INQUIRY SERVICE

Technical consultation is available by calling toll-free in the U.S.A. and Canada.

Ordering, inquiry, customer service, technical .....(800) 445-0345

Domestic fax ordering and inquiry .....(805) 349-2600

International inquiry .....(805) 347-7990

International fax ordering and inquiry .....(805) 349-2607

Website .....www.denmat.com

**MSDS SHEETS AVAILABLE AT [www.denmat.com](http://www.denmat.com)**

751453050W

**DenMat®**  
ISO 9001 CERTIFIED

**1-800-445-0345**