

LUMINEERS® Finishing Kit

Porcelain Finishing Kit



Reorder #033897100

This finishing system is the only one of its kind available to the dental profession. The **LUMINEERS Finishing Kit** includes tools that make **LUMINEERS** placement faster, easier, *better!* This all-inclusive kit allows you to finish **LUMINEERS** without the need for shots or drilling. Each tool is specifically designed to finish the **LUMINEERS** surface and interproximals without causing the patient pain or damaging the **LUMINEERS**.

DIRECTIONS

1. Tooth preparation for **Cerinate®** is an optional procedure due to the strength of the porcelain. Remember, with **Cerinate**, prep if you have to, prep less if you can. Do not prep if it is not necessary.
2. For protrusive prominent line angles or rotated unaligned teeth, prep with diamond bur included in this kit.
3. Assemble the **CeriSaw** and **CeriSander**.
 - a. To **add a strip**, loosen the handle grip by turning counter-clockwise.
 - b. Wrap the **CeriSaw/CeriSander** strip onto the pin located at the front end of the handle, then back around to the pin on the rear end of the handle. The strip forms an "S" shape around the front and back handles. (See Fig. 1)
 - c. Turn the handle grip clockwise to tighten the handgrip, which also tightens the pins and **CeriSaw/CeriSander** strips. (See Fig.2)
 - d. To **remove a strip**, loosen the handle grip by turning counter-clockwise and lift the strip from the pins.

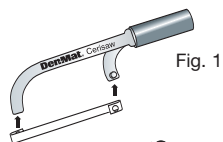


Fig. 1

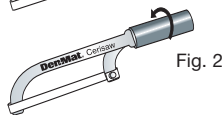


Fig. 2

FINISHING

Note: Maximum bonding occurs in 24 hours. Final finishing is to be performed at next visit.

1. Use **Den-Mat's® Schure 349 instrument** to remove the cured cement along the margins, as well as from the interproximal area along both the facial and the lingual. You may also use bur (No. H246LUF012). This bur is an excellent tool for cleaning up residual resin cement, and also great for finishing composites.
2. Use bur (No. 0856016) to remove the porcelain ledge.
3. Trim/polish the margins of the porcelain veneer. Use bur (No. 858EF014), an extra-fine diamond, to smooth out porcelain.
4. Using the interproximal fine diamond bur (No. 8392016), known as the "mosquito" diamond, refine the margins at interproximal area.

5. Explore the tooth structure next to the **CERINATE LUMINEERS** to check the **LUMINEERS'** margins. There should not be a catch (or bump).
6. If necessary, further polish the margins using the interproximal fine diamond bur again.
7. Check the bite with articulating paper. Use bur (No. 8379018) football-shaped bur on the lingual to finish the incisal wrap, to adjust the high spots or length.
8. Place a wooden wedge interproximally.
9. Use the **CeriSaw** to separate the teeth. Leave the wooden wedge in place.
10. Use the **CeriSander** (fine grit) to smooth the interproximal surfaces.
11. Remove the wooden wedge.
12. Use the **CeriSander** (ultra-fine grit) to further smooth the interproximal surfaces.
It is important that you use each sanding blade only two or three times to open contacts.
13. Check the interproximal surfaces with dental floss for smoothness.
14. Polish your patients new smile with **Porcelain Laminate Polishing Paste**.

STORAGE

Store at room temperature 70° F (23°C)

LIMITED WARRANTY

Den-Mat will replace, or refund the purchase price, of any of its products that are proven to be defective within 30 days of the purchase date. Replacement of defective goods, or refund of the purchase price, shall be the exclusive remedy for the user. Den-Mat will not be liable for any economic, incidental, or consequential loss or damage that arises out of the use of, or the inability to use, its products.

This limited warranty is in lieu of all other warranties, expressed or implied, and shall be void if the product is improperly stored or used. There are no implied warranties of merchantability, fitness for a particular purpose, or otherwise. Before using this product, the user shall determine whether it is suitable for the intended use and the user shall assume all risk and liability associated therewith.

TELEPHONE INQUIRY SERVICE

Technical consultation is available by calling toll-free in the U.S.A. and Canada.

Ordering, inquiry,(800) 445-0345
customer service, technical
Domestic fax ordering and inquiry(800) 349-2600
International inquiry(805) 347-7990
International fax ordering and inquiry(805) 349-2607
Websitewww.denmat.com

Manufactured by Den-Mat Corporation, Santa Maria, CA 93455 U.S.A.

802075900



DenMat®
ISO 9001 CERTIFIED

1-800-445-0345