


# CASE SUBMISSION CHECKLIST

Through extensive analysis of case processing we have found the following to be the four most common reasons for delays and remakes. To save time and money, please educate your entire staff utilizing the checklist below. We have also developed a step-by-step procedure guide outlining the recommended materials and equipment to aid you in case submission.

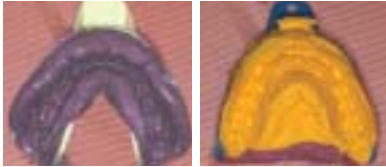

## ITEM 1

Full upper and lower PVS impressions



## ITEM 2 (OPTIONAL)

Models (unmodified)



## ITEM 3

Color copies of patient photos

- Photograph a natural smile
  - Full face front
  - Front view close-up
  - Side 3/4 view (left/right)
  - Occlusal view
- Photograph cheeks retracted
  - Front view close-up
  - Side 3/4 view (left/right)
  - Occlusal view



## ITEM 4

Completed Prescription and Diagnosis Form

- Please insure all areas are filled out.

### MATERIALS AND EQUIPMENT

#### Materials:

- Ultra-Bond Powder and Liquid**—Use **Ultra-Bond** to prevent microleakage polymerization cracks, and pop-offs.  
or **Ultra-Bond Plus Syringeable**—A resin cement formulated specifically to bond to patented **Cerinate Porcelain**.
- Ultra Bond Try-In Paste**—A fail-safe technique. Use this product as a trial run to verify the patient is satisfied with the shade.
- Shade Modification System by Cerinate**—Color tints and opaques in a variety of shades and minimal thickness for customization.
- Tenure Multi-Purpose Bonding System**—Complete etching to enamel system.

#### Equipment:

- Sapphire Plasma Arc Curing Light** with 4mm **Ceri-Taper™ tacking tip**—Provides easy cleanup in the interproximal areas.
- LUMINEERS Magnification System**—Provides superior vision, wide field, and it's lightweight.  
Features:
  - Slim design—lightweight (3.1 oz, 88 g)
  - Wide field
  - Exceptionally bright images
  - Great depth of field
  - Working distance from 14" to 16" (370 to 420 mm)
- CeriSaw and CeriSander**—Interproximal finishing instruments that make life easier. —Tools you'll wonder why you've lived without for so long!
- Schure 349**—Placement instrument for seating LUMINEERS.

### LEARN HOW TO PLACE CERINATE LUMINEERS!

THE PERMANENT, PLEASANT, PAINLESS SMILE

Attend a Cerinate Workshop near you. Call **1-800-445-0345**