


# How to Lengthen Anteriors

## WITHOUT INCREASING VERTICAL DIMENSION

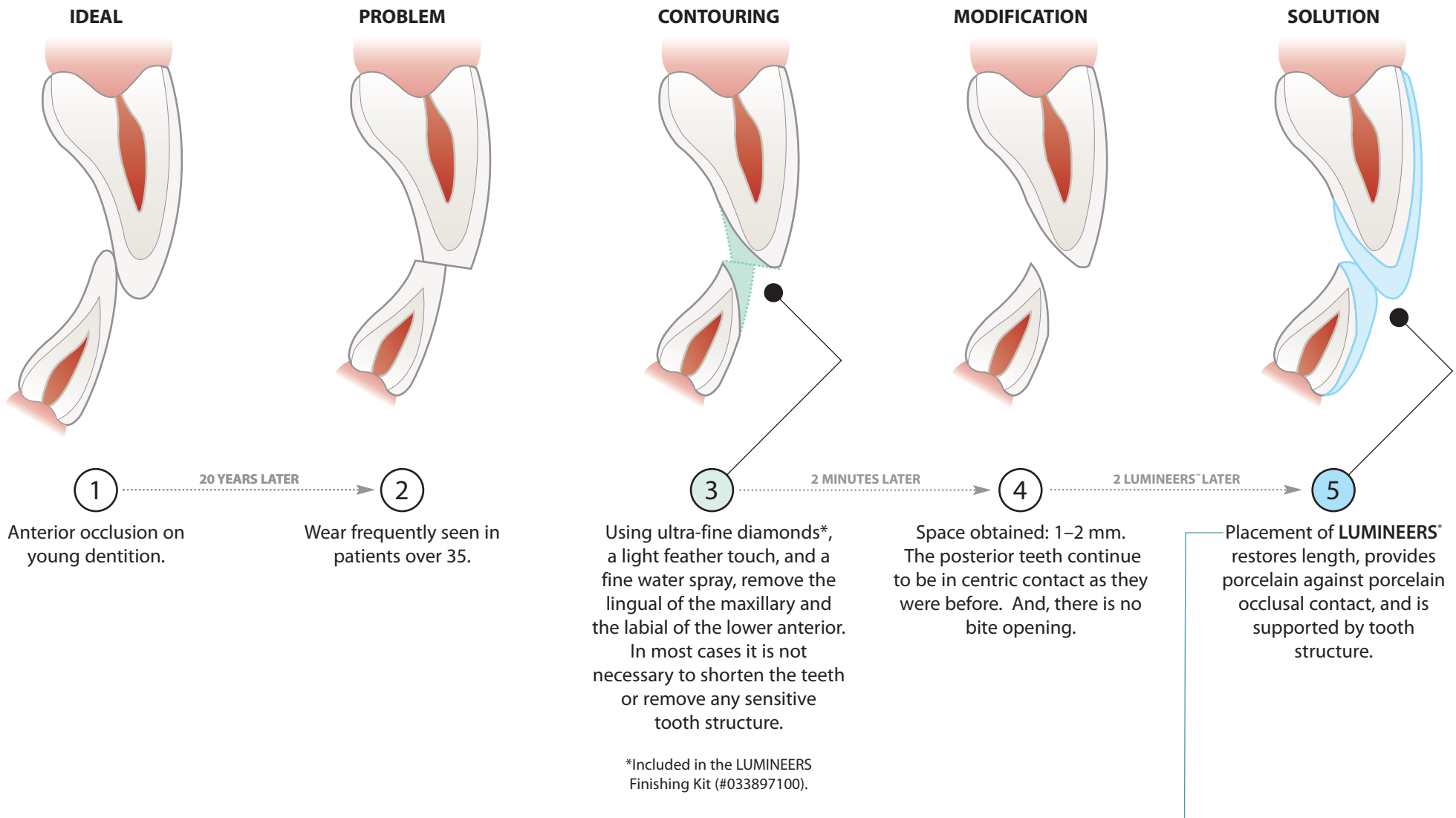

### ★ TIPS

- Create round edges on the maxillary teeth when the porcelain covers the incisal edges.
- When in doubt, send in models and photos for consultation to Cerinate Smile Design Studios. Call 800-872-8384 for Cerinate prescription forms and boxes.

**DenMat**®

**LUMINEERS**®  
BY CERINATE®


# How to Lengthen Teeth Without Increasing Vertical Dimension

—TWO CASE EXAMPLES


## CASE 1



A. Contour guidelines

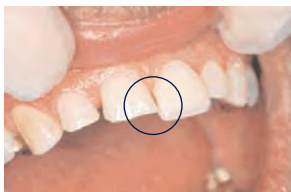

B. Lengthened teeth

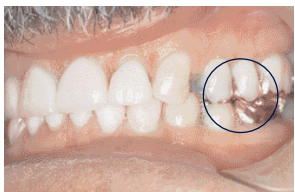

C. Bite did not change



D. Finished result

## CASE 2:



A. Before



B. Cosmetic contour




C. After contour




D. Finished result

See how thin or thick Cerinate LUMINEERS can be.

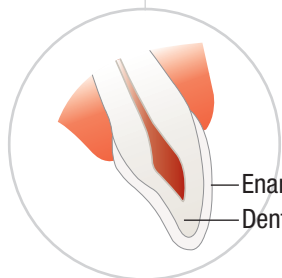

**NORMAL TOOTH STRUCTURE**

Enamel  
Dentin

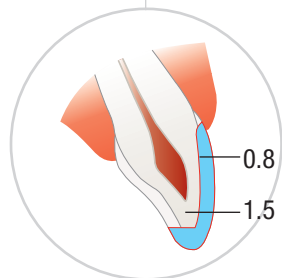


**TRADITIONAL PREP VENEERS**

Approximately 0.8 mm–2 mm healthy tooth structure removal

0.8 mm  
1.5 mm–2 mm


or

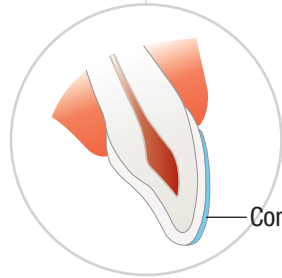

**CONTACT-LENS-THIN LUMINEERS**

No sensitive tooth removal  
0.3 mm–0.5 mm added to the enamel

Contact-Lens-Thin

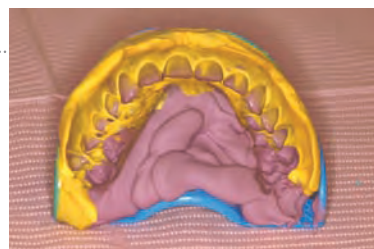

◀ **GET A PERFECT IMPRESSION EVERY TIME WITH LUMINEERS PVS.**

Always pour a model *before* you send the case in to make sure you have an accurate impression that provides sharp margins if you're not using LUMINEERS PVS.



The recommended impression material is **LUMINEERS® PVS.**

Kit No. 031389910

# Adjusting Shades —Avoid Remakes!

Prevent remakes and reappointments by using the Cerinate Shade Modification Kit. With a palette of colors you can simply modify and opaque dark shades right at the chair. There is no need to remake the shade.



*Mask tetracycline stains without removing sensitive tooth structure!*



Cerinate Shade Modification Kit  
No. 033909900



**LUMINEERS®**  
BY CERINATE®